# LavaSurge®

	<p>Read instruction for use. Vor Anwendung Gebrauchsinformation lesen. Вижте инструкциите за употреба. Čtěte návod k použití. Lea las instrucciones de uso. Attention, voir la notice d'instructions Leggere attentamente le istruzioni per l'uso.</p>
	<p>Importer Importeur Вносител Довозце Importador Importateur Importatore</p>

#### GB Wound irrigation solution

**Instruction for Use:**

LavaSurge®– a sterile ready-to-use solution for wound irrigation

**DESCRIPTION**

LavaSurge® is a sterile wound irrigation solutions. LavaSurge® is based on Ringer’s solution and contains Polyhexanide (PHMB) as preservative and the Polyethylene glycol Macroglol 4000.

Polyhexanide prevents the growth of microorganisms in wound dressings and in the solution. Macroglol 4000 optimizes the moistening of the wound surface.

**COMPOSITION**

1 litre contains: Polyhexanide 0.40 g, Macroglol 0.02 g, Ringer solution

**INDICATIONS**

For cleansing and irrigation of acute and chronic non-infected and infected wounds.

For intraoperative wound cleansing and irrigation.

For moistening encrusted dressings and bandages prior to removal.

**APPLICATION**

The entire wound, as well as the surrounding area, should be thoroughly cleansed with LavaSurge®. It is recommended that wounds are soaked with LavaSurge® for a while with a saturated compress in order to achieve optimal cleansing results. The solution should be used undiluted.

**CONTRAINDICATIONS**

LavaSurge® must not be used:

- a) if the patient is known to be allergic to one of the ingredients of the product
- b) on the CNS or the meninges
- c) in the middle or inner ear
- d) in the eyes
- e) on hyaline cartilage
- f) in combination with anionic tensides, cleansing soaps, ointments, oils, enzymes, etc.
- g) for peritoneal lavage or rinsing.

**RESTRICTION OF USE:**

a) Pregnancy and lactation period:

There is no evidence of mutagenic or embryo toxicity associated with the ingredients of this product. Due to the lack of relevant clinical trials and clinical experience with pregnant and breast feeding women, LavaSurge® should only be used after careful medical consultation in these cases.

b) Newborns and infants:

Due to the lack of relevant clinical trials and clinical experience with newborn and infants, LavaSurge® should only be used after careful medical consultation in these cases.

c) Side effects:

Polihexanide can cause allergic reactions such as itching (urticaria) and rashes (exanthema). In rare cases (less than 1 out of 10,000), anaphylactic shock has been reported.

**WARNINGS AND PRECAUTIONS FOR USE**

- Do not use for infusion or injection. Do not swallow.
- Use clear solutions only. Do not use damaged bottles.
- Do not mix with additives.
- Do not re-sterilize! This may critically alter its usability.
- Keep out of reach of children.

**INFORMATION ABOUT SHELF-LIFE**

The expiry date can be found on the label. Do not use after the expiry date!

**AVAILABLE PACKAGE SIZES:**

Plastic bottle PP with screw cap 250 ml, 1000 ml

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