



# Linovera® Scar Repair

GEL WITH HYALURONIC ACID FOR IMPROVED SCAR MATURATION

# Linovera® Scar Repair

Gel with hyaluronic acid for improved scar maturation



*E.g.: Hypertrophic and keloid scar after a Caesarean section*

Did you know that all injuries, burns, or surgical incisions that affect the deepest layers of the skin will result in the formation of a scar?

Once the wound closes, the complex process of reconstruction takes place in layers of the skin.

This **REMODELING** phase is characterized by:

- Intense fibroblast activity.
- Collagen production.
- Synthesis of new blood vessel.

This phase may last from two months to two years.

## DESCRIPTION

Linovera® Scar Repair is a transparent gel composed of:


- Low molecular weight **hyaluronic acid** which hydrates and repairs
- Extracts of **Centella Asiatica** to repair and modulate collagen synthesis
- **Syricalm** that alleviates discomfort



## HOW DOES IT WORK?

**Linovera® Scar Repair's** components enhance skin hydration, alleviate uncomfortable sensations such as itchiness and tightness, and help to soften and flatten scars.

Early and continuous application once the wound has healed can positively influence the maturation process of the scar tissue and contribute to prevent the formation of hypertrophic and keloid scars.

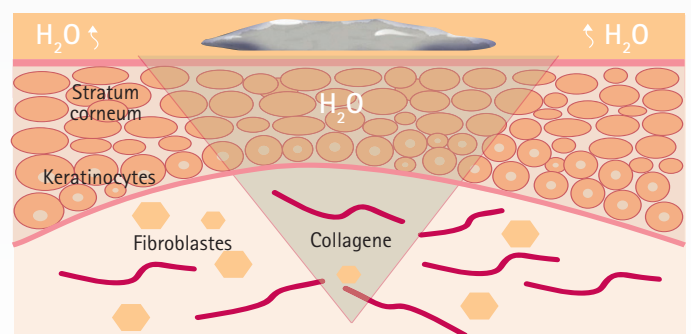

## HOW TO USE?

---

**Linovera® Scar Repair** should only be used on intact skin:

1. Wash the scar and the surrounding skin using a mild soap.
2. Apply the necessary amount to cover the scar with a thin layer of gel.
3. Spread and softly massage to favor absorption.
4. The combined use with **Askina® Scar Repair** dressings is recommended to protect the scar and improve overall efficacy.



## RECOMMENDATIONS:

---

- To achieve optimal results it is recommended to use **Linovera® Scar Repair** day and night, for a minimum period of about 3 to 6 months.
- Begin using **Linovera® Scar Repair** as soon as the wound has healed.
- Apply only on clean and dry skin.

## ADVANTAGES

---

- Transparent gel that is easily applied and rapidly absorbed
- Dermatologically tested
- Hydrates and improves skin elasticity
- Alleviates itchiness and tightness

Linovera® Scar Repair	Size	Pcs/Pack	Article number
-----------------------	------	----------	----------------



15g

1

450450

Askina® Scar Repair	Size	Pcs/Pack	Article number
---------------------	------	----------	----------------



5 x 7,5

5

5695705

4 x 30

5

5694305

10 x 18

5

5691805



2x14

15  
(3 sheets x 5 silicone  
dressing)

5691403

We recommend the combined of Linovera Scar Repair and Askina Scar Repair, soft silicone sheet specially designed for the treatment of scars. Available in different sizes.

Manufactured by:

B. Braun Hospicare Lt. | Collooney | Co. Sligo | IRELAND

B. Braun Medical SA | Carretera de Terrassa 121 | 08191 Rubi (Barcelona) | SPAIN

[www.bbraun.com/wound-management](http://www.bbraun.com/wound-management)

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun.  
Non binding document and photographs.

ZJ01274 / 2019-01