
Ally® Paste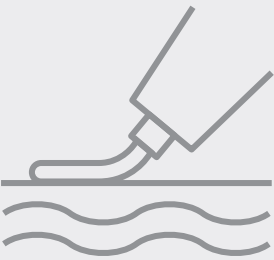

en Instruction for use

1. Product description: Ostomy paste is a convenient and effective skin barrier paste, for use with colostomies, ileostomies, urostomies, it can also be used to fill in spaces caused by skin folds or other irregularities of the skin.

2. Composition: Made of Xanthan gum, PB and PIB.

3. Specification
80g /tube

4. Intended purpose: Paste is intended to fill in cavities/ folds in the intact skin around the stoma. This provides a seal between the stoma and the ostomy base plate and protects the skin against the stoma output.

5. Intended population: The product may be used on individuals with a colostomy, ileostomy or urostomy.

6. Intended user: Health care professionals, caregivers and patients in a hospital or home environment.

7. Indications: For use with a colostomy, ileostomy or urostomy.

8. Warnings:
8.1 Do not use if the tube is damaged or open before use.
8.2 For external use only.
8.3 Do not apply to wounds or damaged skin.
8.4 Should skin irritation occur, contact your doctor or another health professional.
8.5 Do not use on individuals with a known sensitivity to the ingredients.
8.6 Do not use on individuals with impaired immune systems or people receiving immune suppressive treatment.
8.7 Do not use on premature babies, or infants receiving incubator care.
8.8 Do not use if the product expires.
8.9 Discard the tube if the paste appearance changes.

9. Information for use:
9.1 Preparation:
Ensure that the hands and skin around the stoma is clean and dry before applying.
9.2 Application:
9.2.1 Paste may be applied directly from the tube onto the skin, it can also be applied directly on the baseplate. Do not apply paste on the entire adhesive area.
9.2.2 If needed, smooth the paste using moistened cotton swabs or finger.
9.2.3 After applying the baseplate ensure a perfect fit around the stoma by pressing the paste with your fingers.
9.2.4 Remove the residual paste on the tube with a clean tissue. Recap the tube tightly to prevent contact with moisture.
9.2.5 A small amount of paste can be extruded with the help of squeezer.

10. Method of removal: The paste can be removed with lukewarm water. A small amount of residue paste can be left on the skin when applying the next appliance. It is better to leave a slight residue than risk irritating the skin through complete removal.

11. Storage: Keep away from sunlight. Store products at cool and dry conditions – do not freeze.

12. Disposal: The product should be disposed of together with household waste. Do not flush the product down the toilet.

13. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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