Ally® Powder

B | BRAUN

	B.Braun Avitum AG Schwarzenberger Weg 73–79 34212 Melsungen Germany
	Zhende Medical Co., Ltd. Gaobu Town, 312035, Shaoxing, Zhejiang, CHINA
	Shanghai International Holding Corp. GmbH(Europe) Eiffestraße 80, 20537 Hamburg, GERMANY

V22.12.14	
9. Information for use: 9.1 Preparation: Ensure that the skin around the stoma is clean and dry. 9.2 Application: 9.2.1 Unscrew the cap, aim the bottle nozzle at the skin that needs to be taken care of, gently shake the bottle, apply an appropriate amount of powder on the skin around the stoma. 9.2.2 Evenly wipe the powder with cotton swabs, and wipe away the excess powder.	
10. Method of removal: The powder can be removed with lukewarm water.	
11. Storage: Keep away from sunlight. Store products at cool and dry conditions.	
12. Disposal: The product should be disposed of together with household waste. Do not flush the product down the toilet.	
13. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.	

en	Instruction for use
1. Product description: The powder effectively absorbs water, for use with colostomies, ileostomies, urostomies, it can help to protect the skin.	
2. Composition: Made of CMC and Xanthan gum.	
3. Specification 28g/ Bottle	
4. Intended purpose: Ostomy powder is used to absorb moisture from skin around the stoma, which allows for better baseplate adhesion to help protect the skin.	
5. Intended population: The product may be used on individuals with a colostomy, ileostomy or urostomy.	
6. Intended user: Health care professionals, caregivers and patients in a hospital or home environment.	
7. Indications: For use with a colostomy, ileostomy or urostomy.	
8.Warnings: 8.1 For external use only. 8.2 Do not apply to wounds or damaged skin. 8.3 Should skin irritation occur, contact your doctor or another health professional. 8.4 Do not use on individuals with a known sensitivity to the ingredients. 8.5 Do not use on individuals with impaired immune systems or people receiving immune suppressive treatment. 8.6 Do not use on premature babies, or infants receiving incubator care. 8.7 Do not use if the product expires. 8.8 Tighten the cap after each use and store in a cool and dry environment to avoid moisture absorption.	
9. Information for use: 9.1 Preparation: Ensure that the skin around the stoma is clean and dry. 9.2 Application: 9.2.1 Unscrew the cap, aim the bottle nozzle at the skin that needs to be taken care of, gently shake the bottle, apply an appropriate amount of powder on the skin around the stoma. 9.2.2 Evenly wipe the powder with cotton swabs, and wipe away the excess powder.	
10. Method of removal: The powder can be removed with lukewarm water.	
11. Storage: Keep away from sunlight. Store products at cool and dry conditions.	
12. Disposal: The product should be disposed of together with household waste. Do not flush the product down the toilet.	
13. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.	

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