Ally® Rings

B | BRAUN

Ally® Rings

Ally® Rings

Ally® Rings

Ally® Rings

 B.Braun Avitum AG
Schwarzenberger Weg 73-79
34212 Melsungen
Germany

 Zhende Medical Co., Ltd.
Gaobu Town,
312035, Shaoxing,
Zhejiang, CHINA

  Shanghai International
Holding Corp. GmbH(Europe)
Eiffestraße 80,
20537 Hamburg,
GERMANY

 V22.12.14

en Instruction for use

1. Product description: The ostomy ring is intended to fill in cavities/folds in the intact skin around the stoma, for use with colostomies, ileostomies, urostomies, it can help to protect the skin.

2. Composition: Made of release liner and hydrocolloid (CMC, medical hot-melt glue and pectin).

3. Specifications
Size: φ48mm, Thickness: 3mm
Size: φ48mm, Thickness: 4.5mm
Size: φ98mm, Thickness: 3mm

4. Intended purpose: Ostomy ring can be used to fill in or caulk uneven skin contours near the stoma to create a flatter surface and help prevent ostomy drainage from getting under the ostomy baseplate. They may be cut, stretched, or stacked together to help improve the fit of a pouching system.

5. Intended population: The product may be used on individuals with a colostomy, ileostomy or urostomy.

6. Intended user: Health care professionals, caregivers and patients in a hospital or home environment.

7. Indications: For use with a colostomy, ileostomy or urostomy.

8. Warnings:
8.1 Single Use.
8.2 For external use only.
8.3 Do not apply to wounds or damaged skin.
8.4 Should skin irritation occur, contact your doctor or another health professional.
8.5 Do not use on individuals with a known sensitivity to the ingredients.
8.6 Do not use on individuals with impaired immune systems or people receiving immune suppressive treatment.
8.7 Do not use on premature babies, or infants receiving incubator care.
8.8 Do not use if the product expires.

9. Information for use:
9.1 Preparation: Ensure that the hands and skin around the stoma is clean and dry before applying.
9.2 Application:
9.2.1 Apply the ostomy ring to the skin around the stoma.
9.2.2 Adjust the shape again if needed. Mold the ring with your fingers so that the edge of the ostomy ring is securely smoothed onto the skin, or directly paste the ostomy ring on the surface of the baseplate.
9.2.3 Smooth gently with wet fingers if necessary for better adhesion to skin.
9.2.4 Align the center of the baseplate with the opening to ensure a good fit.

10. Method of removal: Removed with lukewarm water. A small amount of residue can be left on the skin when applying the next appliance. It is better to leave a slight residue than risk irritating the skin through complete removal.

11. Storage: Keep away from sunlight. Store products at cool and dry conditions – do not freeze.

12. Disposal: The product should be disposed of together with household waste. Do not flush the product down the toilet.

13. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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