GB

**SwabCap® Luer Access Valve Cap  
with 70% Isopropyl alcohol (IPA) as a disinfectant**

**Available in:**  
Box of 200 single sterile SwabCap® units

**Indications and Usage:**  
SwabCap® is intended for use on swab-able luer access valves as a cover to protect the luer access valves from potential contamination. The SwabCap® acts as a physical barrier to contamination between line accesses and also serves as a disinfecting cleaner for use prior to line access. SwabCap® will disinfect the valve five (5) minutes after application and maintains a disinfected valve surface for up to seven (7) days if not removed.

**Precautions:**  
Do not use if packaging is damaged or opened prior to use. Do not freeze.  
Use by expiration date on package.  
Two-handed removal must be used to make sure luer access valve is not removed unintentionally.

**Instructions for Use:**  
Application  
1. After flush/lock is delivered, inspect the package integrity of the protective covering of the SwabCap® holder.

2. Remove protective cover by pulling tab.  
3. Do not remove SwabCap® from white plastic holder.

4. Hold luer access valve in one hand while pushing and twisting the SwabCap® in a clockwise direction onto the luer access valve.

5. After SwabCap® has been twisted into place, gently pull white plastic holder straight off luer access valve.

6. Verify that SwabCap® has been twisted firmly onto device and that the luer access valve is securely in place. Tighten if necessary.

7. SwabCap® will disinfect the valve after five minutes of application and maintains a disinfected valve surface for up to seven days if not removed. Luer access valves must still be replaced as per institutional guidelines or valve manufacturer's directions for use.

Removal  
1. Two-handed removal must be used to make sure luer access valve is not removed unintentionally.

2. To remove, grasp luer access valve and twist SwabCap® counter clockwise away from valve.

3. Remove SwabCap® and discard.

4. Inspect luer access valve to verify that it is connected securely to the catheter.

5. Valve is ready for access. No further swabbing is necessary.

**Do not use if package is damaged or opened before use. Do not use if product is dry of alcohol. Single use only.**  
Natural rubber latex and preservatives have not been used in the manufacture of this device.

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