

Meliseptol® pure

Fast acting, ready to use
surface disinfection



Meliseptol® pure

- Rapid disinfectant**
for small surfaces
- Schnelldesinfektionsmittel**
für kleine Flächen
- Désinfectant rapide**
pour petites surfaces
- Disinfettante rapido**
per piccole superfici
- Hurtigtvirkende desinfektionsmiddel**
til små overflader
- Szybkodziałający środek dezynfekujący**
do małych powierzchni
- Snabbverkande desinfektionsmedel**
för små ytor
innehåller tensid, har rengörande verkan



ready for
use
1 active within
Minute

perfume
free

FOR MEDICAL DEVICES TIL MEDISKE UDSTYR
FÜR MEDIZINPRODUKTE DO SPÉZIJU MEDICINSKÉHO
POUR DISPOSITIFS MÉDICAUX FÖR MEDICINTEKNISKA PRODUKTER
PER DISPOSITIVI MEDICI



020543 - 2016-05-20

B | BRAUN

Meliseptol® pure

- Rapid disinfectant**
for small surfaces
- Schnelldesinfektionsmittel**
für kleine Flächen
- Désinfectant rapide**
pour petites surfaces
- Disinfettante rapido**
per piccole superfici
- Hurtigtvirkende desinfektionsmiddel**
til små overflader
- Szybkodziałający środek dezynfekujący**
do małych powierzchni
- Snabbverkande desinfektionsmedel**
för små ytor
innehåller tensid, har rengörande verkan

ready for
use
1 active within
Minute

perfume
free

FOR MEDICAL DEVICES TIL MEDISKE UDSTYR
FÜR MEDIZINPRODUKTE DO SPÉZIJU MEDICINSKÉHO
POUR DISPOSITIFS MÉDICAUX FÖR MEDICINTEKNISKA PRODUKTER
PER DISPOSITIVI MEDICI



B | BRAUN

Meliseptol® pure

Fast acting, ready to use surface disinfection

Properties

- Highly efficient – active within 1 minute
- Without additional antimicrobial ingredient (as aldehydes, QAC)
- Perfume-free

Indication

- Meliseptol® pure is a highly efficient disinfectant based on a midlevel alcohol concentration for non-invasive medical inventory, medical devices e.g. examination chairs, hospital beds, theatre equipment and other kinds of surfaces. It is especially suitable for shining metallic and inaccessible surfaces. Meliseptol® pure is aldehyde- and QAC-free and contains only ethanol as antimicrobial active ingredient.

Usage

- Moisten surface completely with undiluted solution for 1 minute.

Physico-chemical data ready-to-use solution

Appearance	clear, colorless liquid
pH-value	approx. 7
Density (20 °C, g/cm ³)	ca. 0.93 g/ml
Flash point (DIN 51 755)	+ 23 °C

Meliseptol® pure – composition:

100 g solution contains: Ethanol 44.0 g, Excipients < 5 % surfactants, purified water, Ingredients in accordance with the Regulations on Detergents EC 648/2004: < 5 % anionic surfactants, Labelling of dangerous goods: see material safety data sheet (MSDS), Cautions: Use disinfectants safely. Always read the label and the product information before use. Do not use the product after the expiry date.

Spectrum of activity Use undiluted under clean conditions at ambient temperature (min. 20°C)	Exposure Time
Bactericidal, yeasticidal with mechanical action (EN 13727, EN 13624, EN 16615)	1 min.
Bactericidal, yeasticidal without mechanical action (EN 13727, EN 13624, EN 13697)**	1 min.
Mycobactericidal (EN 14348)**	60 sec.
Virucidal activity against enveloped viruses (EN 14476)**	30 sec. 60 sec.
Virucidal activity against enveloped viruses incl. HBV, HCV, HIV (DVV/RKI)** ***	30 sec.
Limited spectrum of virucidal activity (EN 14476)** (EN 16777)	30 sec. 5 min.

* the modified test conditions correspond to the requirements of EN 17387

** data for the impregnating solution

*** acc. Recommendation of RKI, Bundesgesundheitsbl. 03-2017

Compatible with the B. Braun Wipes Dispenser System

Meliseptol® pure can be used as impregnating liquid with the B. Braun Wipes Dispenser System. The solution can remain in the dispenser system for up to 4 weeks.



Perfume-free

Product size	REF
250 ml round bottle Meliseptol® pure	19926
1000 ml bottle Meliseptol® pure	19927
5 l canister Meliseptol pure	19958